

MotiliCap Patient Brochure



Understanding Your Digestive Symptoms Starts Here



If you're experiencing persistent nausea, bloating, constipation, abdominal discomfort, vomiting, or other unexplained digestive symptoms, your physician may recommend the MotiliCap™ study to better understand how your digestive system is functioning.

One Simple Test. A Complete Picture of Your Digestive Tract.

MotiliCap is a small ingestible capsule that helps your physician understand how food moves through your digestive system. As the capsule travels naturally through your gastrointestinal (GI) tract, it measures pH, pressure, and temperature, providing valuable information that can help diagnose motility disorders and guide treatment.



Your MotiliCap Journey

At Your Appointment

- Eat the standardized meal provided by your physician.
- Swallow the MotiliCap capsule with water.
- Wear a lightweight recorder that receives information from the capsule.
- After the required fasting period, resume most normal daily activities.

Over the Next 3–5 Days

- Continue your normal routine.
- Wear the recorder throughout the study except while bathing.
- The capsule naturally passes during a bowel movement.

After Your Study


- Return the recorder to your physician's office.
- Your physician will review the results with you and discuss next steps.

Why MotiliCap?

- One comprehensive test evaluates your stomach, small intestine, and colon.
- No radiation exposure.
- Comfortable and non-invasive.
- Continue most normal daily activities.
- Helps your physician develop a personalized treatment plan.

Understanding Your Symptoms

Digestive symptoms such as nausea, bloating, abdominal discomfort, constipation, vomiting, or feeling full too quickly may result from problems with how food moves through your digestive system. MotiliCap provides valuable information about how your GI tract is functioning.

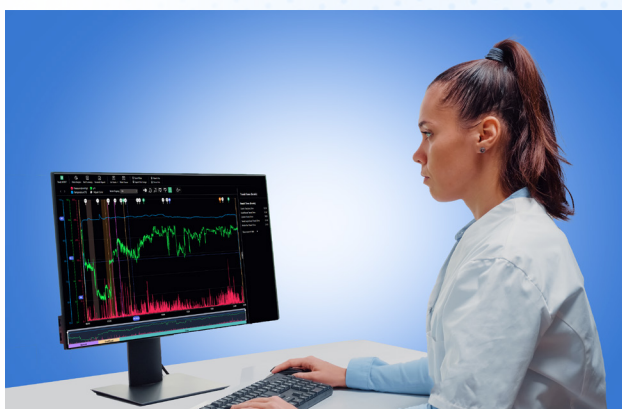

Frequently Asked Questions

How long does the study last?

Your office visit takes about 30 minutes. The study typically lasts 3–5 days.

Will I feel the capsule?

Most patients do not feel the capsule after swallowing it. It is naturally passed during a bowel movement.

How should I prepare?

Do not eat or drink anything except water for 8 hours before your study. After swallowing the capsule, you will fast for an additional 6 hours. Follow your physician's specific instructions.

Can I continue taking my medications?

Discuss all medications with your physician before your study. They will advise which medications should be continued or temporarily stopped.

Important Information

Always follow the preparation and medication instructions provided by your physician. This brochure supplements, but does not replace, your physician's medical advice.

For more information.

If you want more information about your GI condition, talk with your physician about Motility capsule monitoring. Diagnostics will provide beneficial information that can help in establishing a protocol plan that is ideal for you.

Notes

Contact Details



Distributed by:
AnX Robotica
6010 W. Spring Creek Pkwy
Plano, TX 75024
855.777.0020 or 469.606.9495
info@anxrobotics.com

ALWAYS READ THE LABEL. USE ONLY AS DIRECTED. YOUR HEALTHCARE PROFESSIONAL WILL ADVISE YOU WHETHER THIS PRODUCT IS SUITABLE FOR YOU.